

One-Piece Pouch with Soft Drain Tap

For use with high output stomas or draining wounds.

Instructions for Use

Skin should be clean, without traces of soap or grease, and dry before applying the skin barrier.

Cut-to-Fit Skin Barrier

1. Separate front and back pouch panels to avoid cutting the pouch. Cut an opening in the barrier to correspond with the shape and size of the stoma or area to be protected. **DO NOT CUT BEYOND THE LINE INDICATED ON THE BACKING PAPER.**

Applying the Pouch

Before applying the pouch, make sure the pouch drain cap is closed or the pouch is clamped.

2. Remove the release paper from the barrier.
3. Center the opening in the barrier over the stoma or area to be protected.
4. Press the barrier against the skin, working from the center to the edges.

Using the Access Window

5. To close the access window, press firmly around the edge of the access window, supporting the gasket by placing your fingers behind it. A distinctive "snap" indicates the window is securely fastened.

Important: When closing the access window, the edge should be clean. The access window should be completely closed when not in use.

6. To open the access window, separate the plastic tabs on the window.

Draining the Pouch

(Choose one of the options below.)

Using the Pouch Drain

7. Remove the drain cap and drain fluid. Tubing may be attached to the pouch drain and directed into an end collector for continuous drainage.

Using the Drainable Pouch Clamp

Cut off the drain cap from the end of the pouch.

Open the clamp and place it approximately 1 in. (25 mm) from the bottom of the pouch. The finger latch should be pointing toward the top of the pouch.

8. Fold the pouch film over the inner bar of the clamp, making sure there are no wrinkles near the clamp.

Holding the pouch material firmly around the inner bar, close the clamp. Press the two parts together until they lock securely.

Removing the Pouch

9. To remove the pouch, gently peel the barrier away from the skin, starting at the top of the barrier and carefully peeling the barrier downwards until removal is complete.

