

Basic Steps for MicroSkin® Application

MicroSkin® is easy to apply! The MicroSkin® barrier comes with protective paper on BOTH sides. The side strips allow you to handle the barrier without touching the adhesive. Hold the pouch so that you can look at the writing.



1 Fold down the top of each side strip creating a tab for later use. Remove the circular paper in the center if your pouch has a washer.



2 Remove the printed square cover paper from the MicroSkin® barrier.



3 Holding the vertical side strips, apply the pouch over the stoma.



4 Remove the strips of paper from the sidebars by pulling the tabs at the top of each strip, and press down the edges of the barrier.



5 Remove the paper frame from the non-adhesive side of the barrier. Feel the softness and flexibility of MicroSkin®!

