

Instructions for product 1706:

1. Remove spike dust cover and insert spike into appropriate enteral feeding container.
2. Hang feeding container
3. Fill flush bag with desired amount of flush solution
4. Close and hang bag
5. Press Power button on control panel.
6. Open blue door enclosing feeding set loading area.
7. To load feeding set:
 - a. Grasp thumb tab on valve and insert retainer into left pocket
 - b. Grasp black retainer and gently stretch tubing around rotor and insert retainer into right pocket.
 - c. Press down on thumb tab to ensure valve is fully seated.
8. Close blue door
9. Follow screen prompts to prime the pump prior to connecting pump set to patient.
10. Proceed with feeding.