

Instructions for product 1710:

1. Fill bag with desired amount of formula.
2. Close and hang bag. For use with the Kangaroo Joey enteral feeding system, the fluid column in the feeding container must be 6" above the top of the feeding pump.
3. Press Power button on control panel.
4. Open blue door enclosing feeding set loading area.
5. To load feeding set:
 - a. Grasp thumb tab on valve and insert retainer into left pocket
 - b. Grasp black retainer and gently wrap tubing around rotor and insert retainer into right pocket.
 - c. Press down on thumb tab to ensure valve is fully seated.
6. Close blue door
7. Follow screen prompts to prime the pump prior to connecting pump set to patient.
8. Proceed with feeding.