

## **1838 AMT 12IN Bolus Feeding Set STRT PORT Instructions**

### **Instructions:**

1. Inspect contents for damage. If damaged, do not use. Obtain another package.
2. The feeding set may be used for syringe, gravity, or pump feeding or decompression.
3. Attach the feeding set to the MINI button by lining up the dark line on the feeding set connector with the dark line on the MINI button. Press feeding set connector into the MINI button. Turn  $\frac{3}{4}$  to the right (clock-wise) to lock the feeding set in place.
4. When feeding is complete, flush with 5-10ml water. Use a 60ml syringe. Smaller syringes could cause leakage. Remove feeding set by holding the MINI button in place, grasp the feeding connector, twist  $\frac{3}{4}$  turn to the left (counter clock-wise) to unlock, and gently remove the feeding set from the MINI button. Snap MINI button plug in place to keep lumen clean.
5. The feeding set should be cleansed with soap and water and thoroughly rinsed.