

## **2066 Wound Gel Derasyn/ Ag Instructions**

### **Directions:**

1. Cleanse the wound site with sterile water.
2. Apply Derasyn/Ag directly into and around wound area.
3. Cover with a dry absorbent dressing such as DermaRite Bordered Gauze or DermaDress
4. Change daily or as directed by a physician
5. Store at room temperature (59-86 F) DO NOT FREEZE
6. For external use only.



### **Indications:**

- Derasyn/Ag combines the antimicrobial and wound healing benefits of ionic silver, glycerin, and aloe vera with a hydrogel, providing a moist wound environment for maximum healing on partial and full thickness wounds, including:
  - Pressure sores
  - Tissue trauma
  - Surgical incisions
  - Thermal burns
  - Diabetic ulcers
  - Venous stasis ulcers