

**Instructions for use for Replicare Ultra Advanced Hydrocolloid Alginate Dressing:**

- Cleanse the wound with saline solution or an appropriate wound cleanser
- Cleanse and dry the peri-wound skin
- If the peri-wound skin is particularly friable, it may be protected from trauma by applying Skin-Prep
- Choose a dressing large enough to cover the wound with 1/2" of overlap on all sides of the wound
- Remove the printed backing paper exposing the adhesive surface
- Center the dressing over the wound and press the edges firmly to the surrounding skin
- Remove small plastic application tab from underside of the dressing and press all the sides firmly to the skin