




1

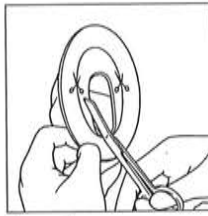

2

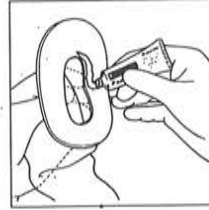


3

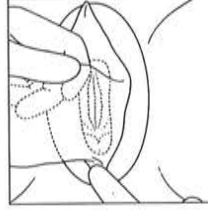


4



5

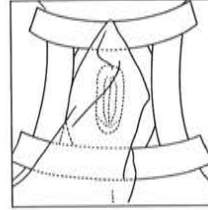

6

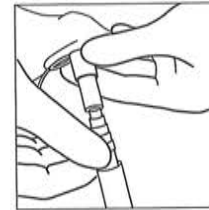

7

US GB Female Urinary Pouch

The one-piece, disposable Female Urinary Incontinence Pouch collects and acts as a conduit for urine by encompassing the vulva. The external pouch is made from odour-barrier film and features a synthetic skin barrier that protects the skin and holds the pouch in place. The pouch should be attached to a receptacle for continuous collection.

WARNING: To help ensure proper adhesion, do not re-use.

Contraindications

Should only be used under supervision of healthcare professional. Use of this product may not be advisable for women with urinary tract, vaginal or other chronic infections, active genital herpes, or chronic urinary retention.

Discontinue use if any of the following symptoms appear: swelling, severe redness, itching or pain in vulva or perineal area, fever, or abnormal vaginal discharge.

Preparation

Position the patient on her back with legs apart and cleanse the external genitalia. Carefully shave the pubic area which will be beneath the skin barrier. Rinse and dry thoroughly. Remove any towel lint that remains on the skin.

Application

Separate the labia (minora and majora) and push back firmly to expose the vulva.

1. Check that the pre-cut opening on the barrier will encompass the urinary and vaginal orifices. This is best done by folding the skin barrier in half and holding it close to the vulva. If it needs to be enlarged mark the backing paper with a pencil.
2. Using scissors, modify the opening in the skin barrier so that it corresponds with the mark on the backing paper. **Do not** cut beyond the line indicated on the backing paper.

Wipe the genital area that will contact the skin barrier with the Skin Protective Film and allow to dry. Be sure the convenience drain cap on the pouch is in the closed position. Remove the release paper from the skin barrier.

3. Around the edge of the opening of the skin barrier, apply a very thin coat of paste, spreading with a damp finger or tongue depressor if necessary. (Note: Too much paste will make the barrier slip on the skin and make positioning more difficult.)

Precautions – Paste

For external use only. Contains alcohol. May sting briefly if applied to irritated or abraded skin. Do not apply paste to severely irritated or excoriated skin or directly over open cuts or sores.

4. With the labia in a normal position, apply the narrow (distal) end of the barrier to the perineum. Gently press the barrier material against the skin working from the perineum upward until the skin barrier contacts the skin at all points.
5. Press the skin barrier against the skin for one full minute to allow the body heat to soften the barrier and enhance its adhesive properties. Allow the patient to assume a normal, comfortable position.
6. Remove the release paper from the strips of microporous adhesive and place the strips on each side of the pouch for additional security as illustrated.
7. Remove the drain cap at the bottom of the pouch and attach the tubing from a bedside receptacle.

Draining the Pouch

If the pouch has been used to collect urine it can be drained by removing the cap on the convenience drain and emptying the urine into an appropriate receptacle. Then replace the cap.

Removing the Pouch

Position the patient on back, legs apart. Remove the tubing and replace the convenience drain cap before removing. Gently remove the microporous adhesive strips and discard. Beginning on either side, ease the skin barrier away from the skin. A water-based jelly may be used to soften the adhesive bond and facilitate removal.

Dispose pouch according to policy. **Do not discard into a sewage system.**

Wash the entire genital area and allow to dry thoroughly before applying another pouch.