

## Instructions for Installing Batteries in Your NX-2000 Non-contact Infrared Thermometer

The NX-2000 non-contact infrared thermometer requires 2 AAA batteries for use (not included). The batteries must be inserted in the correct orientations for the thermometer to function.

1. Slide and lift the purple cap on the base of the thermometer.
2. Insert the 2 AAA batteries into the slots. Place the battery closest to the purple cap positive end (+) up. Place the other battery negative end (-) up. Please see photographs for illustration.
3. Lower cap and slide closed.
4. The thermometer will now be ready for use.

+ Positive end  
- Negative end

