

Directions for Use

- Debride when necessary and irrigate the wound site with normal saline solution.
- Remove excess solution from surrounding skin.
- For heavily exuding wounds, apply to wound bed directly. For wounds with minimal exudate, apply to moistened wound bed; this will initiate the gel forming process.
- Pack deep wounds loosely. The dressing can be cut to size with sterile scissors. The amount of FIBRACOL® Plus dressing to be used depends on the size of the wound and the amount of exudate.
- FIBRACOL® Plus dressing may be covered with either a non-occlusive secondary dressing and fixed to the skin with a non-irritating tape or a semi-occlusive dressing (e.g TIELLE® Hydropolymer Adhesive Dressing or TIELLE® Plus Hydropolymer Adhesive Dressing).
- Reapply FIBRACOL® Plus dressing when the secondary dressing has reached its absorbent capacity or whenever good wound care practice dictates that the dressing should be changed. A heavily exuding wound may require daily or twice daily dressing changes. More moderately exuding wounds will require less frequent changes (every 2 to 4 days or as directed by a healthcare professional).

Following the initial application, irrigate the wound with saline solution. Reapply FIBRACOL® Plus dressing as previously instructed.

Dressing Removal

After gently removing the secondary dressing, lift any FIBRACOL® Plus dressing that has not formed a gel and discard. Using normal saline, gently irrigate the wound to remove any residual gel.

Do not re-use.

Do not resterilize.

Do not use if package is damaged.

The use by date of this product is printed on the packaging.

Caution: Federal Law (USA) restricts this device to sale by or on the order of a properly licensed healthcare practitioner.

This caution is not applicable outside the U.S.