

3M Coban

Self-Adherent Wrap

Non Sterile

(English)

Description: 3M™ Coban™ Self-Adherent Wrap is a laminate of nonwoven material and elastic fibers placed lengthwise to provide elasticity. The elastic wrap contains a cohesive material that makes it stick to itself but not to other materials or skin. The product is supplied in nonsterile and sterile forms.

Indications: Coban self-adherent wrap is intended for use as an elastic wrap to provide compression or support, or to secure dressings or devices. This product is not designed, sold or intended for use except as indicated.



Warnings: As with any elastic wrap, the area of application should be frequently observed for signs of inflammation, discoloration or circulatory deficiency.

Precautions:

1. This product contains natural rubber latex which may cause allergic reactions.
2. Coban self-adherent wrap does not slip or loosen after application. It should be applied with only the appropriate amount of compression.
3. Coban self-adherent wrap should not be placed in direct contact with an open wound.

Directions for Use:

1. Unwind approximately 12 inches of Coban self-adherent wrap from the roll and allow the wrap to relax.
2. Without stretching, apply one full wrap and overlap. Press overlapped area lightly to keep the end in place. See Figure 1.
3. Proceed with wrapping, applying tension to the wrap only to obtain desired compression. Always unwind more Coban self-adherent wrap from the roll before making a wrap so that the product is not applied with too much tension. See Figure 2.
4. When application is complete, cut off excess Coban self-adherent wrap. Secure the end of the application by gently pressing.

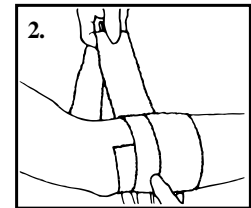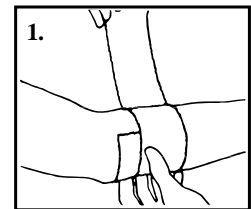

Storage and Shelf Life: This product should be stored at room temperature, 59-86°F (15-30°C). For best performance, store Coban self-adherent wrap in its individual package until use. For shelf life, refer to the date printed on each package.

Ordering Information: Supplied in boxes of individually packaged rolls. All rolls are 5 yards (4,5m) long stretched.

If you have any questions or comments, please contact the 3M Health Care Customer Helpline at 1-800-228-3957.

Explanation of Symbols:

⚠ • Attention, see instructions for use.

LOT ⌚ 2010-10 AZ • The lot in a box and the hourglass symbols are symbols that represent lot number and expiration date. The hourglass is followed by a year and month which represent the expiration date (year and month: 2010-10). The entire line after the hourglass represents the lot number (2010-10-AZ).

3M

Made in U.S.A. by
3M Health Care
St. Paul, MN 55144-1000
(U.S.A.) 1-800-228-3957
www.3M.com

© 3M 2005
3M and Coban are
trademarks of 3M
38-9018-6171-6