

SURFACE DISINFECTION – CAVICIDE®

SURFACE DISINFECTION USING CAVICIDE:

Spray ➡ Wipe ➡ Spray

STEPS 1, 2, 3 – CLEAN BEFORE DISINFECTING



1. Spray CaviCide directly onto surface to be cleaned.



2. Wipe with clean paper towel to remove debris and bioburden.



3. Discard towel.

Use CaviCide to clean and disinfect:

- Potentially contaminated noncritical instruments and equipment.
- Hard, nonporous surfaces or items to prevent cross contamination, such as:
 - Light fixtures
 - Clinical and laboratory surfaces
 - Telephone and office machines

For a complete listing of uses, go to metrex.com.

STEPS 4, 5, 6 – DISINFECT AFTER CLEANING



4. Spray CaviCide directly onto precleaned surface, thoroughly wetting area. Follow label instructions for appropriate contact times.



5. Wipe with clean paper towel.



6. Discard towel.

PART NO. US / Canada	PRODUCT DESCRIPTION	PACKAGING
13-1002	CaviCide – 2 fluid ounce spray	48 bottles/case
13-1008	CaviCide – 8 fluid ounce spray	12 bottles/case
13-1024 / 11-1024	CaviCide – 24 fluid ounce spray	12 bottles/case
13-1000 / 11-1000	CaviCide – 1 gallon	4 bottles/case
13-1025 / 11-1025	CaviCide – 2.5 gallon	2 bottles/case

Note: Follow standard precautions for personal protection as appropriate.



For more information, visit metrex.com/education, or call 800.841.1428.

Lit #77-1004-3