

Instructions for Use
Mode d'emploi

FLEXI-TRAK[®] Large/Grand Modèle

Anchoring Device
Dispositif de fixation

Single Use Only
A Usage Unique



INDICATION

For anchoring tubes and catheters to skin, such as: urinary catheters, gastrostomy tubes, I.V. tubes, rectal tubes, chest tubes, central venous lines, pacemaker wires, sump tubes, biliary drainage tubes, nephrostomy tubes, and peritoneal dialysis tubes.

CONTRAINDICATION

Use on patients with a known sensitivity to acrylic adhesive or its components.⁽¹⁾

INSTRUCTIONS FOR USE

Ensure area is clean, dry and free from powder and greasy substances.

Remove the plain white backing paper from the device.

Apply the body pad adhesive to the desired location.

Remove printed backing paper from reusable tape on the top side.

Place the catheter/tube along the center of the device.

Mold both sides of the tape to the contour of the catheter/tube. Check that tape is secure around catheter/tube.

To remove **FLEXI-TRAK** Large device: remove catheter/tube then apply light pressure on adjacent skin while gently pulling the adhesive from the skin.

Notes:

- 1) Apply only to intact skin.
- 2) Can remain in place up to 5 days.
- 3) The tabs are resealable to accommodate repositioning of the catheter/tube.
- 4) Accommodates catheter French sizes 8-36. Shorter wear times have been experienced with indwelling urinary catheters due to increased body movement. **FLEXI-TRAK** Large Anchoring Device is indicated for use with indwelling urinary catheters French sizes 18-36.¹
- 5) Contact with water has been known to weaken the skin adhesive in **FLEXI-TRAK** Large, resulting in decreased wear time.
- 6) Due to the inherent characteristics of silicone, clinical trials have shown that the resealable tapes of **FLEXI-TRAK** Large may not adhere firmly around silicone and silicone-coated catheters.⁽¹⁾

Adhere to local disposal requirements/regulations.

Avoid refrigeration and exposure to high humidity.

ConvaTec[®] offers a complete line of ostomy products. For more information on ConvaTec products, contact your health care professional, ostomy supply dealer, or call 1-800-422-8811 (US only).

In Canada only, call
1-800-465-6302.

How Supplied

Flexi-Trak[®] Anchoring Device

LARGE (4" x 1 1/2")

50 anchors/box Nonsterile

Order No. 37449

SMALL (2 1/2" x 1 1/2")

50 anchors/box Nonsterile

Order No. 37448

⁽¹⁾Data on file, ConvaTec

©2010 ConvaTec Inc.