



# AllKare®

## Protective Barrier Wipe

Instructions for Use

 **Convatec**



Read instructions for use.



Single use

Provides a barrier film layer on skin under ostomy skin barriers, adhesives, tapes, and hydrocolloid dressings.

### PRECAUTIONS

**FOR EXTERNAL USE ONLY.** Avoid contact with the eyes. Do not use directly on wound, burn or stoma. Use only around wound, burn or stoma.

**WARNING:** Use only as directed. Keep out of reach of children. Do not use near open flame.


### DIRECTIONS:

1. Cleanse area, rinse, dry thoroughly.
2. Apply AllKare® Protective Barrier Wipe to area requiring protection and allow to air dry. In the event of contact with excoriated skin, momentary stinging may be experienced.
3. Apply ostomy skin barrier, adhesive, tape, etc.
4. Upon removal of skin barrier, adhesive, etc., remaining residue can be removed with AllKare® Adhesive Remover.

### DISPOSAL

Adhere to local disposal requirements/regulations.

### STORAGE CONDITIONS

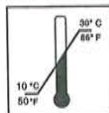

Storage temperature. Avoid excessive heat.



Lot/control number



Expiry date

Note: Should irritation occur, discontinue use and consult a healthcare professional.