
PRODUCT DESCRIPTION

The Sur-FitTM Irrigation Adapter Faceplate allows use of the Sur-Fit Irrigation Sleeve by individuals using Active Life or other one-piece pouches.





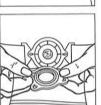

- 1. Attach a Sur-Fit ostomy belt to one side of the faceplate. (Do not attach the belt to the irrigation sleeve flange).
- Position the faceplate opening over the stoma and while holding in place bring the belt around the waist and attach to the other side of the faceplate. While holding the faceplate in position, adjust the belt to fit snuggly.
- Locate the bottom lip portion on the back of the irrigation sleeve and the faceplate flange, and starting at the bottom, apply gentle pressure around the circumference of the sleeve until it "snaps" into place and feels securely attached. A downward tug on the sleeve will determine if proper flangeto-flange locking has taken place.
- Irrigate following the instructions in the Sur-Fit Irrigation Sleeve insert.

HOW SUPPLIED

The Sur-Fit Irrigation Adapter Faceplate* is available in two flange sizes:

	(Diameter)	Order No.
	45 mm (1 3/4 ")	22742
(\bigcirc)	70 mm (23/4")	22744



*Must be used with the Ostomy Appliance Belt.

The following products can be used with the faceplate:



VISI-FLOW Irrigator includes a calibrated bag, a flow indicator, flow regulator and a pliable stoma cone

22735



SUR-FITTM Irrigation Sleeve** Box of 5 sleeves

45 mm (1 3/4 ") 70 mm (23/4")

24252 24254

22710



Sur-Fit® Flange.

1 tail closure **Must be used in combination with the Sur-Fit Irrigation Adapter Faceplate NOTE: The irrigation sleeves are available in three other flange sizes (38 mm, 57 mm and 100 mm) for use with the Stomahesive® Wafer with

HOW SUPPLIED (Con't)	Order No.
Ostomy Appliance Belt adjustable Box of 1 belt	175510
The following products are also available:	
ACTIVE LIFE® One-piece Closed-End Ostomy Pouch Opaque Transparent Box of 30 Pouches	22703 22706

ACTIVE LIFETM One-piece

Stoma Cap Opaque Box of 30