



1)



2)

DIRECTIONS FOR USE:

- 1) Remove the peel strip from adhesive.
- 2) Position the elasticized cup end of the guard to the back of a pair of close-fitting underwear. The front end of the guard should be even with the front waistband. Press guard into underwear, adhesive side down. Pull underwear into place and adjust guard so that it cups the testicles. Change as often as you feel comfortable.

MODE D'EMPLOI

- 1) Enlevez la bande de l'adhésif.
- 2) Placez l'extrémité de la coquille élastisées de la protection à l'arrière d'un sous-vêtement ajusté. L'avant de la protection doit être aligné sur la ceinture avant. Pressez la protection dans le sous-vêtement, côté adhésif vers le bas. Enfilez le sous-vêtement et ajustez la protection de sorte qu'elle enveloppe les testicules. Remplacez la protection lorsque nécessaire pour vous sentir à l'aise.

MODO DE USO

- 1) Retire la banda protectora del adhesivo.
- 2) Ponga el extremo elástico de la copa del protector en la parte posterior de una prenda de ropa interior que se ajuste bien. El extremo frontal del protector debe estar alineado con la parte frontal de la pretina. Presione el protector contra la ropa interior, con el lado del adhesivo hacia abajo. Hale la ropa interior para colocarla en su lugar y ajuste el protector de manera que encajen los testículos. Cámbielo con la frecuencia necesaria para sentirse cómodo.