

# Stomahesive® Strips

## Moldable Adhesive

### Instructions for use




Single use



Read instructions for use.

#### STORAGE CONDITIONS

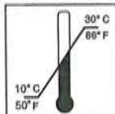

Storage temperature.



Keep dry. Avoid high humidity.




Lot/control number

#### STOMAHESIVE® STRIPS


For use as a filler to help protect uneven skin surfaces due to scarring, skin folds or depressions which are adjacent to an ostomy, fistula, or draining wound; or for the attachment of a male external catheter.

**NOTE:** The skin **MUST** be free from any solvent or greasy substance.

#### FOR OSTOMY USE

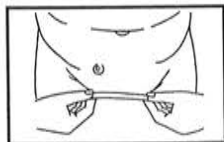


1. Cut the Stomahesive strip to the length of the skin fold, scar or crevice; several strips may be used to "layer" the depression.

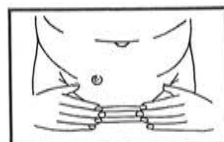


2. Remove the backing paper from only **one** side of the strip and press into place. (If "layering" the strips, remove the backing paper from **both** sides of the strips.)
3. Remove the outer backing paper.
4. Apply the ostomy wafer over the Stomahesive Strips, according to the use instructions of the wafer.

#### FOR USE WITH MALE EXTERNAL CATHETERS

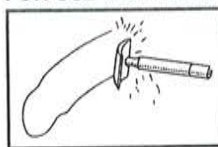


1. Remove excess hair if necessary.

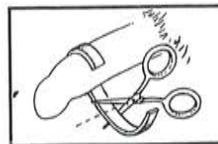


2. Cut the Stomahesive Strip to a length which is slightly longer than the circumference of the penis.




3. Remove the silicone release paper from one side of the strip only and attach the strip midway up the shaft of the penis, and no closer than 1" from the scrotal area.

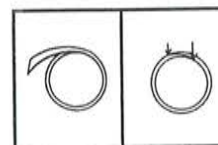

4. Press well into place allowing one or two minutes for good adhesion.

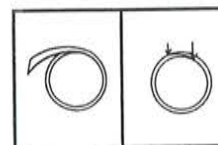

5. Before completely removing the outer release paper, press the overlapping ends of the adhesive mass together to form a continuous sealing ring as shown.

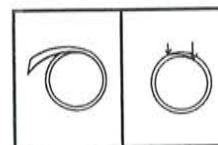

6. Remove the remaining release paper and attach the male external catheter as described in that product's instructions for use.

Manufactured by:



Convatec Inc.  
Skillman, NJ 08558  
1-800-422-8811

Convatec Limited, Deeside, CH5 2NU, UK

Distributed by:  
Convatec Canada Ltd.  
Montréal, Québec, Canada  
1-800-465-6302

[www.convatec.com](http://www.convatec.com)

© 2008 Convatec Inc.  
®/™ indicates a trademark of Convatec Inc.