### Product Description

The Sur-Fit Night Drainage Container Set is comprised of a plastic graduated container (2000 cc capacity), tubing with an attached Accuseal® connector and a cloth cover. The container set is designed for use with the Sur-Fit® Urostomy Pouch with Accuseal® Tap. However, also included is a universal adapter for attaching the tubing to a regular Sur-Fit® Urostomy Pouch or other urostomy pouches.

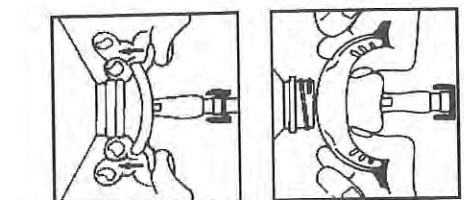
### GENERAL INSTRUCTIONS

1. Keep the tubing from twisting or kinking so the urine can easily flow into the container.
2. Wash and rinse the drainage container and tubing daily with mild soap and water.
3. Prior to connecting the tubing to a pouch for night drainage, a small amount of water may be added to the container to keep it in one position.


4. Remove the protective cap from the Accuseal connector at the free end of the tubing by pulling apart while squeezing the green arms, prior to attachment to a pouch.

### ATTACHING THE CAP TO THE CONTAINER

1. Roll the side tabs and bottom lip of the cap back towards the tubing.

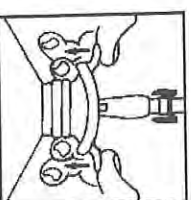
2. Rest the cap in the container opening.

3. Roll the side tabs down until the cap snaps into place.


4. To remove the cap, pull back on the side tabs.
- NOTE:** The tubing can be removed from cap by pulling apart while squeezing the small green arms.

### ATTACHING TO A SUR-FIT UROSTOMY POUCH WITH ACCUSEAL TAP

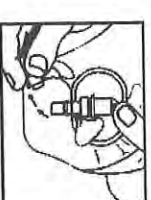
1. With the pouch's swivel valve in the closed position, remove the plug from the tap.

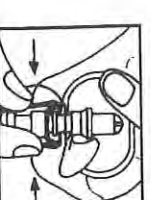2. Position the Accuseal connector of the tubing over the tap stem and push until they snap in place.

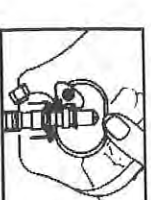
3. Turn the swivel valve to the open position. The red dot should be facing away from the body.


1. Close the swivel valve on the pouch. The red dot should be facing toward the body.


### REMOVING THE CONTAINER SET FROM A SUR-FIT UROSTOMY POUCH WITH ACCUSEAL TAP

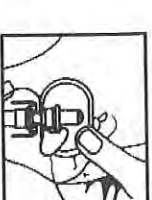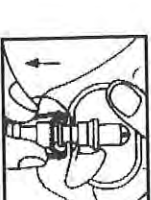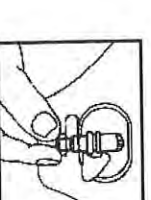2. While holding the drainage assembly, pull apart, squeezing the small green arms.

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