


# UltiCare™

Safety Syringe

**Providing a premium quality, low-cost alternative that protects patients and health care professionals.**

## SAFETY

Simply load the safety syringe with medication, slide the safety shield over the needle and transport the syringe safely to its destination, administer, slide, twist and click and the syringe is ready for proper disposal.

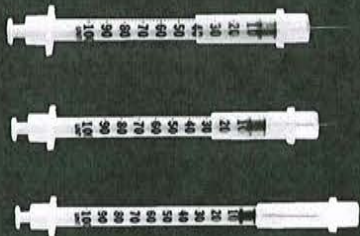

**Bold markings are easily read through the safety shield.**

## FUNCTION

### 1 **Preparation**

Draw up and administer medication following established protocol.

### 2 **Post Injection**

After injection, extend safety shield forward until a click is heard.

### 3 **Lock**

After safety shield is fully extended, simply twist it in either direction until you hear a click. The needle is now fully protected.



## SAVINGS

**Start adding up the savings with each injection.**

UltiCare Safety Syringes low-dead space design reduces medication waste and expense.

### **Low Dead Space.**

Our needles are attached directly to the syringe barrel. The elimination of the hub reduces medication waste and accurately and precisely delivers each dose.

**UltiCare™**