

Instructions for Transparent Opsite:

Step 1

- Wash hands before and after treating the wound
- Gently clean the wound removing any debris and dry the surrounding skin. Ensure surrounding skin remains dry before application. Do not apply to heavily bleeding wounds.
- Remove dressing from pouch.
- Remove the backing paper marked 1, taking care not to touch the adhesive side.
- Place the dressing over the wound using light pressure.

Step 2

Remove the backing paper marked 2, and smooth the dressing down. Press around the edges of the dressing.

Step 3

If required, record the wound size.

Step 4

Remove the green grid and keep in patient's records.

Step 5

To remove the dressing, lift one corner and slowly stretch the film in a motion that is parallel to the skin. Handle and dispose of the used dressing with care.

Please see Overview tab for Contraindications/Precautions