

[Print](#) [Close](#)

EXU-DRY WOUND DRESSING

Anti-Sheer Absorbent Dressing



Step 1

Using a full absorbency EXU-DRY^α dressing, select a size slightly larger than the wound

Step 2

Place onto wound. The back side of the dressings are marked "Use other side against wound".

Step 3

EXU-DRY^α dressings may be used as a Wet or Dry dressing:

- Dry: a variety of topical agents may be applied directly on the wound or buttered on to the dressing.
- Wet: submerge in liquid topical agent and gently squeeze out (do not ring out) excess fluid.
- Secondary dressing: May be used in combination with other products (calcium alginates, gels, foams, etc.) that have been used to pack a wound.

Step 4

light wrap of roll gauze.

2010 SMITH & NEPHEW
