

## **EN** Product Description

ADAPTIC® Non-Adhering Dressing is a primary dressing made of knitted cellulose acetate fabric and impregnated with a specially formulated petrolatum emulsion. It is designed to help protect the wound while preventing the dressing from adhering to the wound.

- Helps prevent dressing adherence
- Helps protect regenerating tissue and minimizes patient pain and trauma during dressing changes
- Helps prevent the pooling of fluid at the wound site
- Exudate easily passes through to the secondary absorbent dressing
- Can be cut to wound size without unravelling and shredding

### **Indications**

ADAPTIC® Non-Adhering Dressing is indicated for dry to highly exuding wounds where adherence of dressing and exudate is to be prevented including:

- 1st & 2nd degree burns
- Abrasions
- Grafts
- Venous ulcers
- Pressure ulcers
- Nail extractions
- Eczema
- Staples
- Surgical incisions
- Lacerations
- Reconstructive procedures
- Suture lines

### **DIRECTIONS FOR USE**

#### **Site Preparation**

1. Prepare the wound according to wound management protocol.
2. Ensure skin surrounding the wound is dry.

#### **Dressing Application**

1. Carefully clean the wound following facility protocol.
2. Cut ADAPTIC® Dressing to size if required and place directly on the wound.
3. Cover with sterile secondary dressing of your choice depending on the amount of wound exudate.
4. ADAPTIC® Dressing may be left in place for several days depending upon amount of exudate.

**Do not use if package is damaged.**

**Do not reuse.**

**Do not resterilize.**

**The use by date of this product is printed on the packaging.**

**Caution:** Federal Law (USA) restricts this device to sale by or on the order of a properly licensed healthcare practitioner.

*This caution is not applicable outside the U.S.*