

[Print](#) [Close](#)

ACTICOAT[®]◇

Antimicrobial Barrier Dressing



ACTICOAT[®] offers a range of premium antimicrobial dressings to help in the prevention and reduction of infection in a variety of wounds

[read more](#)[Overview](#)[Instructions for Use](#)

Step 1



Remove ACTICOAT[®] from the package and moisten the dressing with sterile water (do not use saline).

Wring out excess water prior to applying.

Step 2



Cut to appropriate size and shape.

Step 3



Apply the dressing to the wound.

2010 SMITH & NEPHEW
