

[Print](#) [Close](#)

BIOSTEP◇

Collagen Matrix Dressing



Step up the healing process. BIOSTEP◇ overcomes barriers to wound closure for difficult to heal wounds with it's 6-day wear time and unique, dual-action MMP targeting and deactivation.



Overview



Instructions for
Use

Step 1

Prior to application, the wound bed should be debrided and then irrigated with an appropriate wound cleanser or water.

Step 2

The dressing should be cut to fit the wound size exactly. For heavily exudating wounds, the dressing should be applied directly to the wound bed. For dry wounds with minimal exudate, the wound bed should be moistened with water to begin the gelling process.

Step 3

BIOSTEP◇ should be covered with a suitable secondary dressing.

Step 4

2010 SMITH & NEPHEW
