



for a free sample, visit
MULTIDEXSAMPLES.COM

restrictions apply. see website

directions for use

preparation of site

1. Necrotic tissue should be debrided according to acceptable practice or as directed by an attending physician.
2. The site should be irrigated liberally with a sterile physiological 0.9% normal saline or a balanced salts solution.

application of multidex powder or gel

1. After irrigation, apply Multidex®.
 - a. For shallow wound – ¼" thick over entire wound site
 - b. For deep wound – fill wound site to surface taking care to fill all undermined areas
2. Cover with a non-adherent, non-occlusive dressing such as Covaderm Plus®, MultiPad™, Sofsorb®, or Polyderm™ Plus. If necessary, tape in place or use roll gauze or Stretch Net™ to secure dressing.
3. Dressing change should be once a day on minimally to moderately draining wounds and twice a day on heavily exuding wounds.

The specially designed tube makes application of Multidex Powder or Gel easy.

Multidex Powder can be covered with a non-adherent, non-occlusive Covaderm Plus dressing.



Patented

removal of multidex

1. Remove non-adherent dressing with care. If dressing adheres to wound, soak with saline for several minutes before removing so the fragile granulation tissue is not disturbed.
2. Flush site liberally but gently with a sterile, physiological irrigating solution to remove debris. This will leave newly formed granulation tissue undisturbed.
3. Usual frequency of dressing change is once a day, depending upon drainage and the type of secondary dressing.

available in a variety of sizes

Multidex Powder and Multidex Gel are supplied sterile. To order a smaller case quantity, add -1 after the product number.

Example: 46-703-1.

Product #	Size	Qty/Bx	Qty/Cs
Multidex® Powder			
46-704	6 grams	10	60
46-703	12 grams	5	30
46-702	25 grams	5	30
46-701	45 grams	6	36
Multidex® Gel			
46-711	¼ fl. oz.	10	30
46-710	½ fl. oz.	10	30
46-712	3 fl. oz.	6	36
46-713	1 fl. oz.	3	30

TO PLACE AN ORDER OR FOR MORE INFORMATION
 PLEASE CALL: 800.251.9864

865.938.7828 (OUTSIDE USA)
 800.543.2182 (FAX)

For clinical information about our wound care treatment system, visit our web site at www.deroyal.com.

DeRoyal®, DeRoyal logo, Improving Care. Improving Business.®, Covaderm Plus®, and Sofsorb® are registered trademarks of DeRoyal Industries, Inc.
 MultiPad™ and Polyderm™ are trademarks of DeRoyal Industries, Inc.