

ComfortFoam™

FOAM WOUND DRESSING
WITH SOFT SILICONE ADHESIVE

For the management of exuding wounds

DESCRIPTION

ComfortFoam is a self-adherent silicone foam wound dressing consisting of a soft silicone contact surface, a flexible, absorbent polyurethane foam pad and a vapor-permeable, moisture-proof outer film. The silicone adhesive sticks to surrounding skin but not to the wound bed, minimizing pain and the risk of damage to the wound and periwound areas.

INDICATIONS FOR USE

For the management of moderate to heavily exuding partial or full thickness wounds, including: pressure ulcers, diabetic ulcers, arterial ulcers, venous ulcers, traumatic wounds.

CONTRAINDICATIONS

ComfortFoam should not be used on people who are sensitive or allergic to silicone and/or any other contents of the dressing.

DIRECTIONS (Please use under the guidance of a healthcare professional.)

PREPARE WOUND SITE AND SURROUNDING SKIN:

1. Cleanse wound with advised liquid to remove residue according to local infection control protocol. Deep wounds should be well irrigated.
2. Skin around wound should be clean and dry, use skin barrier prep as needed.

TO APPLY:

1. Remove dressing from package.
2. Dressing may be cut to size prior to application.
3. Remove backing paper from dressing.
4. Apply sticky side of dressing to moist wound bed overlapping 1-2 inches of dry skin; gently smooth into place.
5. Secure edges with tape if desired.

ComfortFoam™

FOAM WOUND DRESSING
WITH SOFT SILICONE ADHESIVE

For the management of exuding wounds

CHANGING THE DRESSING:


- Dressing may be left in place for up to 7 days.
- Change the dressing if drainage seeps through the back of the dressing, or if the edge of the dressing becomes loose; or if the condition of the wound or surrounding skin causes concern; or as directed by a qualified healthcare professional.
- Dressing may be moistened to ease removal.
- Carefully remove adhesive dressings by holding the edge of the dressing and slowly pulling it parallel to the skin.
- Dispose of in accordance with local guidance.

WARNINGS

- For single use only.
- If signs of infection are detected, a qualified healthcare professional should be contacted to decide on the next course of action.
- If signs of an allergic reaction are detected, discontinue use of the product and seek advice from a qualified healthcare professional.
- Product is sterile unless the package is opened or damaged. Do not resterilize.

OTHER INFORMATION

- Store in a dry place at room temperature.
- Keep away from sunlight.
- Not made with natural rubber latex.

DEPTH / STAGE	ABSORBENCY
PARTIAL FULL 1 2 3 4	 HIGH
For Partial and Full thickness wounds / Pressure ulcers stages 2-4	

DermaRite®