DermaBlue[™] + Foam Transfer

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For the management of exuding wounds

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DESCRIPTION

DermaBlue+ Foam Transfer is a thin foam wound dressing that is highly hydrophilic. DermaBlue+ Foam Transfer is made of polyurethane/polyether foam, Methylene Blue (up to 0.25 mg/g), Gentian Violet (up to 0.25 mg/g), and Silver Sodium Zirconium Phosphate (up to 7 mg/g). The 3 ingredients provide broad spectrum antimicrobial protection within the dressing.

INDICATIONS FOR USE

DermaBlue+ Foam Transfer is indicated for moderately exuding, partial to full thickness wounds, including: pressure ulcers, leg ulcers, diabetic foot ulcers, graft wounds and donor sites, skin tears, first and second degree burns, surgical wounds, lacerations and abrasions.

CONTRAINDICATIONS

DermaBlue+ Foam Transfer should not be used on patients with a known sensitivity to silver or any other contents of the dressing. Not indicated for third-degree burns.

DIRECTIONS (Use under the guidance of a qualified healthcare professional.)

PREPARE WOUND SITE AND SURROUNDING SKIN:

1. Cleanse wound to remove residue according to local infection control protocol.
2. Skin around wound should be clean and dry, use skin barrier prep as needed.

TO APPLY:

1. Remove dressing from package.
2. Dressing may be cut to size prior to application. Dressing may be rolled or layered to lightly pack into wound.
3. Apply dressing directly to wound. Direct contact with entire wound bed is necessary for best results.
4. Secure with tape or appropriate cover dressing that manages drainage and maintains a moist wound environment.

CHANGING THE DRESSING:

- Dressing may be left in place for up to 3 days or as directed by a qualified healthcare professional.
- Change the dressing if drainage shows through the back of the cover dressing; or if the edge of the cover dressing becomes loose; or if the condition of the wound or surrounding skin causes concern; or as directed by a qualified healthcare professional.
- Dressing may be moistened to ease removal.
- Carefully remove adhesive secondary dressings by holding the edge of the dressing and slowly pulling it parallel to the skin.
- Dispose of in accordance with local guidance.

WARNINGS

- For single use only.
- If signs of infection are detected, a qualified healthcare professional should be contacted to decide on the next course of action.
- If signs of an allergic reaction are detected, discontinue use of the product and seek advice from a qualified healthcare professional.
- Product is sterile unless the package is opened or damaged. Do not resterilize.

OTHER INFORMATION

- Store in a dry place at room temperature.
- Keep away from sunlight.
- Not made with natural rubber latex.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician (or properly licensed healthcare practitioner).

DEPTH / STAGE	ABSORBENCY
PARTIAL FULL	
1 2 3 4	MEDIUM
For Partial and Full thickness wounds / Pressure ulcers stages 2-4	

DermaRite[®]

DermaBlue[™] + Foam Transfer

ANTIMICROBIAL FOAM WOUND DRESSING WITH METHYLENE BLUE, GENTIAN VIOLET, AND SILVER

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