

compared to Biatain Silicone Foam Dressing.

- Biatain Foam Dressings are sterile single-use, soft, highly absorbent and conformable polyurethane foam dressings that provide an optimal moist wound healing environment and effective exudate management.
- Biatain Foam Dressings are protected with semi-permeable film backings that are waterproof and provide bacterial barriers.

## How to use Biatain Silicone and Biatain Silicone Lite Foam Dressings

### Application



1. Rinse the wound with lukewarm water or physiological saline. Gently dry the skin around the wound. The safe use of other cleansing agents, in combination with Biatain Foam Dressings, has not been demonstrated.
2. If any film, cream, ointment or similar product is used, allow the skin to dry before dressing application.
3. Select a dressing where the foam pad overlaps the wound edge by a minimum of 1-2 centimetres
4. Remove Biatain Foam Dressings aseptically from the packaging. Do not touch the plain (non-printed) side of the foam.
5. Use the handles to prevent touching the dressing and to ensure aseptic application. Remove the centre protective film first and place the foam part of the dressing over the wound. Then remove the release films on the sides as part of the dressing application.
6. Biatain Foam Dressings should be changed when clinically indicated or when visible signs of exudates approach the edge of the dressing. The dressing should not be left in place for more than 7 days without clinical inspection by a physician.

### Removal



1. To remove the dressing, gently lift the corners of the dressing away from the wound. It is recommended to loosen the adhesive border before lifting the dressing away from the wound.

### Disposal

1. Do not flush the products down the toilet.