

## Removal

To remove the dressing, gently lift the corners of the dressing away from the wound.

When removing adhesive dressings you are advised to loosen the adhesive border before lifting the dressing away from the wound.

Biatain Adhesive Foam Dressing/Sacral should be removed from the top edge and down towards the anus to minimise the risk of transmitting infection.

## Directions

### Biatain Foam Dressings

1. Rinse the wound with lukewarm water or physiological saline. Gently dry the skin around the wound. The safe use of other cleansing agents in combination with Biatain Foam Dressings has not been demonstrated.
2. If any film, cream, ointment or similar product is used, allow the skin to dry before applying the dressing.