

3. Remove Biatain Foam Dressings aseptically from the packaging. Do not touch the plain (non-printed) side of the foam.
4. Apply Biatain Foam Dressings according to the illustrations and application directions. Select a dressing where the foam pad overlaps the wound edge by at least 2 centimetres (an overlap of 1 centimetre is sufficient in the case of small dressings).
5. Biatain Non-Adhesive Foam Dressings and Biatain Soft-Hold Foam Dressings must be secured with a secondary bandage or compression bandage.
6. Biatain Foam Dressings should be changed when clinically indicated or when visible signs of exudate approach the edge of the dressing.
7. Biatain Foam Dressings may be left in place for up to 7 days depending on the amount of exudate, dressing condition and type of wound.

Biatain Foam Filler/Cavity (Same directions as above point 1 and 2).

1. Apply Biatain Foam Filler/Cavity according to the illustrations and application directions.
2. For fixation and covering use a separate dressing, e.g. Biatain Adhesive Foam Dressings/Biatain Adhesive Foam Dressing/Sacral dressings (for highly exuding wounds) or Comfeel Plus Ulcer Dressings/Comfeel Sacral Dressings or Comfeel Plus Transparent Dressings (for moderately exuding wounds).
3. You are advised to change Biatain Foam Filler/Cavity when the secondary dressing is changed. Ensure that Biatain Foam Filler/Cavity is completely removed from the wound bed.