



1

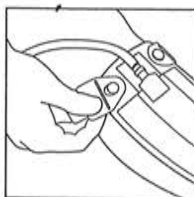


2



3a

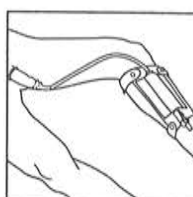

3b

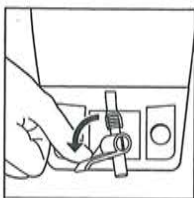

4

## **US GB Instructions for Use**

Single patient use only.

### **Applying the Leg Bag**

If the leg bag has a cap on the tap, remove it.

1. Close the drain valve at the bottom of the bag.

Place the leg bag on either the inner or outer portion of the calf or thigh.

### **Applying the Extension Tubing**

2. Determine the appropriate length for the extension tubing. The tubing should span the distance between the catheter and the leg bag comfortably without tugging. Remove any excess tubing with scissors and discard.

Insert the tubing into the bottom of the catheter. Moistening the connector usually eases insertion.

Attach the free end of the extension tubing to the connector at the top of the leg bag. *Make sure extension tubing is not kinked or obstructed.*

### **Applying the Fabric Leg Bag Straps**

- 3a, 3b. Attach a leg bag strap to one of the buttons at the top of the leg bag. Bring the strap around the leg and attach it to the corresponding button. Repeat this procedure with the other strap. **Note:** The strap's ribs must be worn next to the skin in order to keep the bag stable. Adjust the straps so they are comfortable and secure. *To ensure adequate circulation, do not overtighten. Check the area under the straps periodically for skin irritation or trauma.*

### **Draining the Leg Bag**

4. To drain the leg bag, open the drain valve and empty the contents into a toilet or other receptacle. Close the drain valve to reseal the leg bag.

The leg bag, tubing, and straps are reusable. The leg bag and extension tubing should be cleaned daily. The straps are machine washable and should be air dried.