

## Product Description


Features a patented green locking device that securely positions the Foley catheter on patient's upper thigh or waist, as possible.

## Application Instructions

1. Position leg band high around patient's thigh or on patient's waist with Dale product label pointing to inside of leg. Stretch leg band in place and fasten hook and loop tab. Proper compression has been achieved when two fingers fit snugly under the band.
2. Place Foley catheter over white stitching of green locking device. Leave an ample loop in catheter above leg band to avoid bladder traction; or stretch catheter for traction when required.
3. With catheter in desired position, insert the narrow, green tab over catheter and through the square opening so that green tabs overlap.
4. Lock catheter in position by pulling green tabs in opposite directions. Secure tabs in place. To reposition catheter, raise either green tab, adjust catheter, and refasten tab.
5. Another method of securement is to place the Y-port of the proximal end of the catheter onto the locking device and secure.

## Additional Benefits

When green, meatal irritation, leg band can be rotated in either direction. Alternating legs periodically may also add to greater patient comfort.

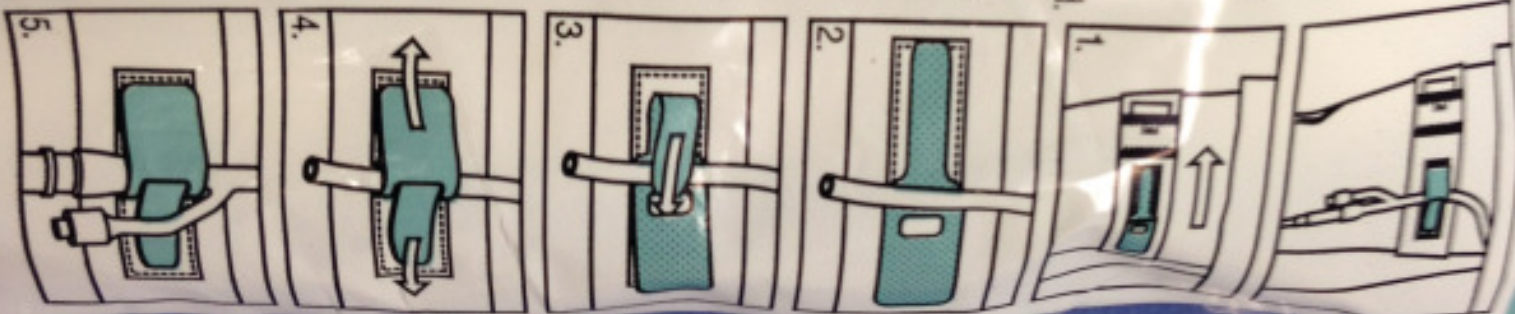

FOR SINGLE PATIENT USE