

# Magellan™ Insulin and Tuberculin Safety Syringe

## Instructions For Use

1



Select the appropriate Magellan™ Insulin or Tuberculin Safety Syringe for your procedure.

2



Open package by peeling back paper tabs and remove syringe.

3



Remove protective needle sheath.

4



Draw up medication and administer injection using an aseptic technique in accordance with your facility's protocol.

5



Lock the safety shield using your thumb



...your finger.

6



Verify locked position through audible and tactile "click". Locked position will completely encase needle.

7



Once the shield is locked, immediately dispose of syringe in an approved sharps disposal container.



**COVIDIEN**

*positive results for life™*

COVIDIEN, COVIDIEN with logo and  
™ marked brands are trademarks of Covidien AG or its affiliate.  
©2007 Covidien AG or its affiliate. All rights reserved. Patents Pending.

H-6176-5M-0108

15 HAMPSHIRE STREET  
MANSFIELD, MA  
02048

1.800.962.9888  
508.261.8000

[www.covidien.com](http://www.covidien.com)