

# Magellan™ Insulin and Tuberculin Safety Syringe

## Instructions For Use



1  
Select the appropriate Magellan™ Insulin or Tuberculin Safety Syringe for your procedure.



2  
Open package by peeling back paper tabs and remove syringe.




3  
Remove protective needle sheath.



4  
Draw up medication and administer injection using an aseptic technique in accordance with your facility's protocol.



5  
Lock the safety shield using your thumb

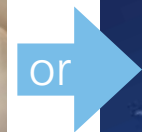

...your finger.



6  
Verify locked position through audible and tactile "click". Locked position will completely encase needle.



7  
Once the shield is locked, immediately dispose of syringe in an approved sharps disposal container.



**COVIDIEN**

*positive results for life™*

COVIDIEN, COVIDIEN with logo and ™ marked brands are trademarks of Covidien AG or its affiliate. ©2007 Covidien AG or its affiliate. All rights reserved. Patents Pending.

H-6176-5M-0108