

Magellan™ Insulin and Tuberculin Safety Syringe

Instructions For Use



1
Select the appropriate Magellan™ Insulin or Tuberculin Safety Syringe for your procedure.



2
Open package by peeling back paper tabs and remove syringe.



3
Remove protective needle sheath.



4
Draw up medication and administer injection using an aseptic technique in accordance with your facility's protocol.



5
Lock the safety shield using your thumb



...your finger.



6
Verify locked position through audible and tactile "click". Locked position will completely encase needle.



7
Once the shield is locked, immediately dispose of syringe in an approved sharps disposal container.



COVIDIEN

positive results for life™

COVIDIEN, COVIDIEN with logo and ™ marked brands are trademarks of Covidien AG or its affiliate. ©2007 Covidien AG or its affiliate. All rights reserved. Patents Pending.

H-6176-5M-0108

15 HAMPSHIRE STREET
MANSFIELD, MA
02048

1.800.962.9888
508.261.8000

www.covidien.com