

## To install the cap

The Luer-Lock Tip syringe includes a screw-in lock to keep the needle securely attached. Connect the instrument by twisting the Luer-Lock Tip onto a sterile Luer-Lock syringe.



## To Remove the cap

Place the needle cap on the bed or table with the hand that is not holding the syringe. Be careful to keep the needle away from anyone, including yourself.

Guide the needle into the cap, using just one hand. Do not use your other hand for assistance.

Pick up the cap with the syringe, so the capped needle is pointing straight up. Carefully grasp the cap near the needle hub, sliding the cap towards the syringe. Twist the cap and needle combination counterclockwise to remove them.

Twist the capped needle counterclockwise, grasping it low near the hub, to loosen it from the syringe. Dispose of the needle immediately by placing it in a sharps container.