

Fixodent[®] Powder Adhesive Instructions

To apply powder adhesive:

- 1) Clean dentures
- 2) Wet dentures
- 3) Apply powder in thin layer as shown on the bottle or carton
- 4) Shake off all loose powder
- 5) Insert Dentures and hold briefly in place



DO NOT use more than $\frac{1}{4}$ teaspoon. Shake off excess. If powder comes off denture in your mouth you are using too much.

DO NOT use more than once a day. With proper use this bottle should last at least 9 to 10 weeks.

DO NOT use excess powder for poorly fitting dentures.

Consult your dentist regularly to ensure you have properly fitting dentures. Poorly fitting dentures may impair your health.

WARNING: DO NOT use more than directed. Contains zinc. Excessive and prolonged zinc intake is reported to be associated with serious health problems. Consult a doctor if using other products containing zinc.