


Fixodent Adhesive Cream Original Instructions

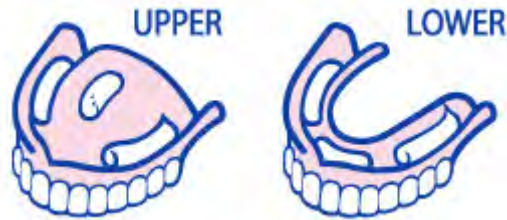

To apply adhesive:

- 1) Clean & Dry Dentures
- 2) Apply Adhesives in thin strips as shown
- 3) Insert Dentures and hold briefly in place

DO NOT use more product than shown in diagram (for full dentures, not more than 6 strips or about 3 inches total length). For Fixodent Food Seal, please refer to the application instructions and diagram on the carton. If product oozes off denture in your mouth, you are using too much.

DO NOT use product more than once a day.

DO NOT use excess product for poorly fitting dentures.

Consult your dentist regularly to ensure you have properly fitting dentures. Poorly fitting dentures may impair your health.

WARNING: DO NOT use more than directed. Contains zinc. Excessive and prolonged zinc intake is reported to be associated with serious health problems. Consult a doctor if using other products containing zinc.