




Print Close

ALLEVYN◇ GENTLE BORDER

Gel Adhesive Hydrocellular Foam Dressing with Border

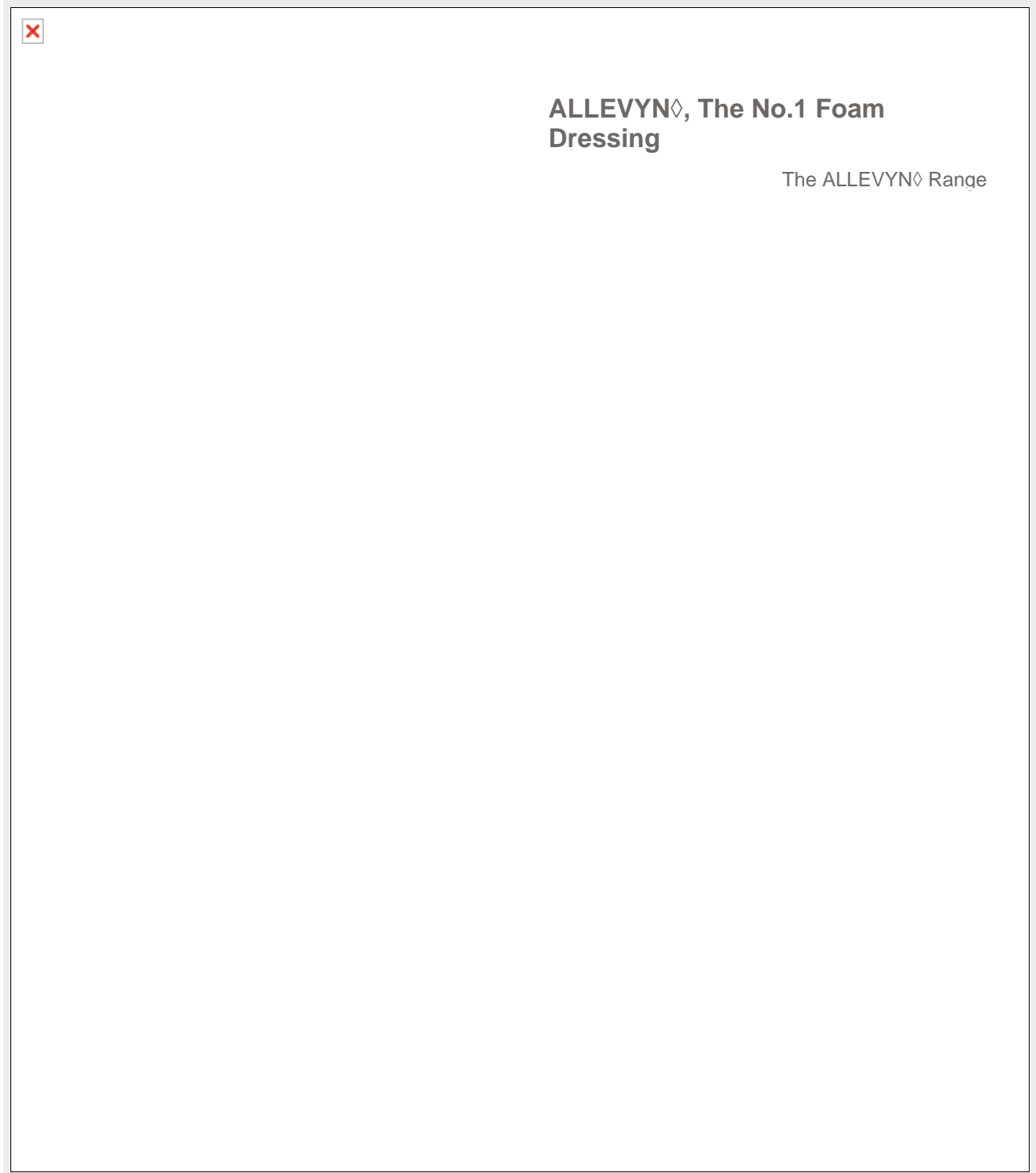



Step 1

Select appropriate dressing size.

Step 2

Prepare and clean the skin surrounding the wound area by removing excess moisture. Any excess hair should be clipped to ensure close approximation to the wound.

Step 3

Start to remove the protector material from ALLEVYN \diamond Gentle Border and anchor the adhesive side of the dressing to the skin.

Step 4

Smooth the dressing over the wound removing the remaining protector material and ensure that the dressing is adhered all around the wound.

Dressing changes

During the early stages of wound management ALLEVYN \diamond Gentle Border dressings should be inspected frequently. Where the product is used on infected wounds the infection should be inspected and treated as per local clinical protocol. Dressings can be left in place for up to 7 days depending on the condition of the wound and the surrounding skin or until exudate is visible and approaches to within 5/8" of the edge of the dressing, whichever is sooner.

2010 SMITH & NEPHEW
