

[Print](#) [Close](#)

## ALLEVYN<sup>®</sup> AG ADHESIVE

Adhesive Absorbent Silver Barrier Dressing



**ALLEVYN<sup>®</sup>, The No.1 Foam Dressing**



Overview



Instructions for  
Use

### Step 1

Select an appropriate size dressing. Prepare and clean the skin around the wound. SKIN-  
PREP<sup>®</sup> Protective Wipes may be used to protect fragile skin. Peel back the protector papers and  
anchor the dressing over the wound. Smooth the dressing into place, ensuring that the edges of  
the dressing are not wrinkled.

### Step 2

During the early stages of treatment, ALLEVYN<sup>®</sup> Ag Adhesive dressings should be inspected  
frequently; dressings can be left in place until exudate can be seen through the outer waterproof  
film layer and approaches 3/4" from the edge of the dressing.

### Step 3

To remove, lift one corner of the dressing and lift it gently away from the wound.

**Please see Overview page for Precautions**

2010 SMITH & NEPHEW

---