



en

Please read and be thoroughly familiar with the contents of this document prior to using the Monoject Tube Holder.

1. Open pouch using aseptic technique. Remove protective cap from male luer fitting of Collection Set.
2. Insert multiple sample luer adapter into the female luer connector of the Monoject Tube Holder as shown in Figure 1.
3. Perform venipuncture and fill blood culture tubes and/or blood collection tubes with desired volumes of specimen.
4. Dispose of contaminated holder set and Monoject Tube Holder according to individual facility policy into an appropriate Sharps container.

Under no circumstances should the user tamper with, or attempt to separate components after use. This device is not to be recycled.