

## **GB** Instructions for use

Intended for use with implanted intravascular ports.

Not intended for use with gastric band ports.

**Instructions for Use: Use Aseptic Technique.**

- Follow CDC guidelines and/or institutional protocol for use and replacement.

### **Insertion of WHIN Infusion Set**

1. Locate the implantable port by palpating the patient and disinfect skin per institutional protocol.
2. Firmly attach female connector to fluid source and prime set to expel all air.
3. Remove needle protector by pulling straight off.
4. Insert needle into septum of implantable port at approximately 90° angle. Ensure that the needle has fully penetrated the septum.
5. After penetrating septum, check for patency via aspiration. Resistance to aspiration may indicate partial or complete needle occlusion. Discontinue use of needle if resistance is encountered.
6. Secure set with sterile dressing, fill port, and begin therapy per institutional protocol.

### **Removal of WHIN Infusion Set**

1. When the therapy is complete, flush port per institutional protocol. Terminate flow using positive pressure technique.
2. Stabilize the port by placing two fingers on the patient. Remove by pulling straight up on the wings of the WHIN Infusion Set.
3. Dispose of the WHIN Infusion Set in a sharps container.

Rx only