

EasyMax LTC & NG Glucose Meter Control Solution Test (CST) · Quick Reference Guide

Step 1



You will need the meter, a test strip, control solution, and a tissue.

Step 2



Put a test strip into the meter in the direction of the arrow. The meter will automatically turn on.

Step 3



Place the control solution between your fingertips and roll the solution back and forth.

Step 4



Wipe off the top of the control solution cap with a tissue.

Step 5



Squeeze out one to two drops of control solution and discard.

Step 6



After you discarded the first drops, squeeze out another drop of control solution onto the control solution cap.

Step 7



Touch the drop to the blood collection area at the end of the test strip. The strip will sip in the control solution. The meter will count down from 5 seconds to get the results.

Step 8



Do not remove the test strip yet. Check if the reading falls within the range printed on the test strip bottle.

ERROR 1



CAUTION 1

DO NOT apply the control solution directly from the control solution bottle into the strip! This can cause cross contamination with the enzyme from the strip and the entire control solution bottle will no longer be accurate on future tests.

ERROR 2



CAUTION 2

DO NOT apply the control solution or capillary whole blood on TOP of the strip. This will lead to an inaccurate reading or no reading at all.

If you have any questions regarding control solution testing please call OakTree's Customer Service Line at 866-994-3345.

718D031313_X01