

## Instructions for Use

Medlance plus® is very easy to handle and its intuitive contact activation minimizes training required for new users.

### Step 1

#### 1. Twist

Twist off protective cap and then pull it straight out.

### Step 2


#### 2. Press

Press the Medlance® plus body firmly against the puncture site to activate the device.

### Step 3

#### 3. Collect

Gently apply intermittent pressure near the puncture site to obtain the required blood volume.

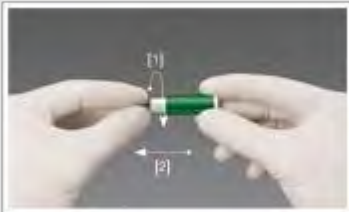

#### 1. Twist

**Twist off protective cap and then pull it straight out.**



#### 2. Press

**Press the Medlance® plus body firmly against the puncture site to activate the device.**



#### 3. Collect

**Gently apply intermittent pressure near the puncture site to obtain the required blood volume.**

## Recommendations for blood collection test:

Patients should wash their hands with warm water and soap before the test.

Remember that warm fingers increase the blood flow.

Fingertip test site should be selected slightly off-center to minimize any pain and maximize blood flow.

The patient's arm should hang down below the elbow to allow blood flow to fingertips.

After the blood collection, use a tissue and apply firm pressure directly over the finger-stick site.

Discard used lancet into a sharps container in accordance with facility guidelines, national and state regulations.

**Medlance plus® is very easy to handle and its intuitive contact activation minimizes training required for new users.**