



PASSION. PARTNERSHIP. POSSIBILITIES.

# ADVANCED WOUND CARE



# Wound Closure Strips



## Description

DUKAL Wound Closure Strips can be used in conjunction with or as an excellent alternative to sutures or staples. Each DUKAL quality strip consists of thin non-woven reinforced fibers that are arranged in a bilateral pattern for high tensile strength, to provide incision support and maintain closure. This adhesive backed product is translucent, porous, and breathable. Similar to Steri-Strips®.

## Features & Benefits

- Can be used in place of sutures or staples
- Quick, simple application
- Non-invasive, less physical trauma
- Comfortable to wear
- Improved esthetic results; no puncture marks




## Indications:

- Provide support following early suture/staple removal
- Close wounds without significant tension, edema, stretch or movement
- Secure devices to skin (such as IV)
- Secure skin grafts in burn patients
- Secure skin flaps


## Contraindications

- Infected wounds


## Easy Application:




Thoroughly clean and dry the skin around the wound.




Open pouch and remove contents using aseptic technique.




Remove tab.



Apply closure at wound mid-point.



Apply additional closures for larger wounds.



Apply closures every 3mm and additional closures parallel to wound margin.

Item	Description	Packaging
Sterile - Latex Free		
5150	1/8" x 3"	5/pk, 50 pk/bx, 4 bx/cs
5151	1/4" x 3"	3/pk, 50 pk/bx, 4 bx/cs
5152	1/4" x 1 1/2 "	6/pk, 50 pk/bx, 4 bx/cs
5156	1/4" x 4"	10/pk, 50 pk/bx, 4 bx/cs
5157	1/2" x 4"	6/pk, 50 pk/bx, 4 bx/cs
5158	1" x 4"	4/pk, 25 pk/bx, 4 bx/cs
Item	Description	Packaging
Non-Sterile - Latex Free		
C-02	1/4" x 3"	500/bx
C-03	1/4" x 1 1/2 "	500/bx
C-04	1/4" x 4"	500/bx
C-05	1/2" x 4"	500/bx
C-06	1" x 4"	500/bx

# Transparent Semi-Permeable Dressings



## Description

The DUKAL Transparent Dressing, with easy two step application is an excellent dressing for IV sites, minor burns, and wounds. They keep contamination out while allowing for moisture vapor transfer. Similar to Tegaderm®.

## Features & Benefits

- High Moisture Vapor Transmission Rate (MVTR)
- Waterproof
- Bacteria proof
- Conformable
- Transparent film allows for easy monitoring of site
- Micro porous for breathability
- Rounded edges to prevent roll up
- Removable patient label for chart
- Easy 2 step application process



# Indications:

- I.V., Donor, Incision, and Vaccine Sites
- Minor burns (1st and 2nd degree)
- Lacerations & abrasions
- Partial thickness- wounds
- Skin tears
- Stage I or II pressure ulcers
- Autolytic debriding (not appropriate for wounds that have become infected)
- Skin biopsies
- Protective eye coverings
- Post tattoo application and removal


# Contraindications

It is not recommended that the material be applied over deep cavity wounds, third degree burns, or wounds that show evidence of clinical infection. The dressing should not be used to retain in-dwelling arterial catheters


# Change Frequency

The frequency of dressing changes will be governed by the state of the wound, but can be left for up to 7 days depending on amount of wound drainage


# Easy Application:




Wash and dry hands thoroughly.

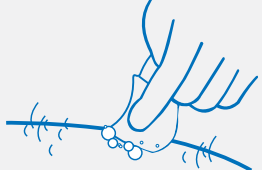


Prepare skin for application.



Open package and remove sterile dressing.



Holding perimeter paper by end tabs, position and apply dressing adhesive -side down. Smooth and seal the dressing from the center to the edge.



Firmly smooth the edge prior to removing the paper frame.

Item	Description	Packaging
Sterile - Latex Free		
4045	1 1/2" x 2"	1/pk, 100 pk/bx
4067	2 3/8" x 2 3/4"	1/pk, 100 pk/bx
4112	4" x 4 3/4"	1/pk, 50 pk/bx
4112	6" x 8"	1/pk, 10 pk/cs

# Caliber™ Non-Woven Island Dressings




## Description

A convenient all-in-one dressing with a soft non-woven adhesive border and non-adherent pad. It's gentle on skin and absorbent providing the perfect combination of protection and comfort.

## Features & Benefits

- Highly conformable - flexes with movement or swelling
- Easy to apply
- Excellent adhesion but gentle on skin
- Water resistant non-woven backing
- Soft, cushioned non-adherent pad protects wound from trauma

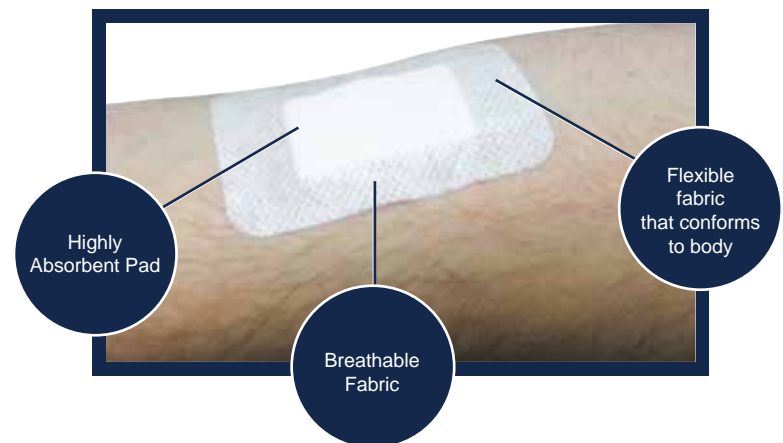

## Indications/Common Applications:

- Light to moderately draining wounds
- Superficial & partial thickness wounds
- I.V. catheter sites
- Incision sites
- Cuts and lacerations
- To secure or cover devices

Item	Description	Packaging
Sterile - Latex Free		
4070	2" x 3"	50/bx, 8 bx/cs
4071	4" x 5"	25/bx, 8 bx/cs
4072	4" x 8"	25/bx, 4 bx/cs
4073	4" x 10"	25/bx, 4 bx/cs
4074	4" x 14"	25/bx, 2 bx/cs
4075	4" x 4"	25/bx, 8 bx/cs
4076	6" x 6"	25/bx, 4 bx/cs

## Connect with us!

Find our full line of  
products online at:

**[www.dukal.com](http://www.dukal.com)**



### Corporate Office

DUKAL Corporation  
2 Fleetwood Court,  
Ronkonkoma, NY 11779

Telephone: 1-631-656-3800

Toll Free: 1-800-243-0741

Fax: 1-631-656-3810

Email: [customerservice@dukal.com](mailto:customerservice@dukal.com)

To Place Orders: [orders@dukal.com](mailto:orders@dukal.com)