

5970 Female Catheter Kit Instructions and Ingredients

Directions:

Contents arranged in sequence:

1. Unfold CSR wrap and use as a sterile field
2. Don gloves
3. Pull catheter out of tube to desired length. Lay tube on sterile field.
4. Prepare patient and proceed with catheterization. (**Note:** If urine does not flow freely into the specimen tube, loosen the cap slightly.)
5. After sample collection, pull catheter out of top; tighten cover and depress spout.
6. Send to lab in normal manner.

Aplicare Three Povidone-Iodine Swabsticks

Drug Facts	
Active ingredient	Purpose
Povidone-iodine USP 10%	Antiseptic
Use	
Antiseptic skin preparation	
Warnings	
Do not use: <ul style="list-style-type: none">• If allergic to iodine• In the eyes For external use only Ask a doctor before use if injuries are: <ul style="list-style-type: none">• Deep or puncture wounds• Serious burns Stop use and ask a doctor if: <ul style="list-style-type: none">• Redness, irritation, swelling, or pain persists or increases• Infection occurs Avoid pooling beneath patient Keep out of reach of children. In case of accidental ingestion, seek professional assistance or consult a poison control center immediately.	
Directions	
Apply locally as needed	
Other information	
<ul style="list-style-type: none">• 1% titratable iodine• Latex free• For hospital or professional use only• STERILE unless opened or damaged	
Inactive ingredients	
Citric acid, disodium phosphate, nonoxynol-9, sodium hydroxide, water	

Dover Lubricating Jelly 5g

- Water-soluble lubricant.
- For hospital and professional use.
- Will not harm surgical instruments, rubber, or plastic.
- Non-irritating to skin, tissue, and mucous membranes.
- Do not use if package is opened or damaged.