

## 5970 Female Catheter Kit Instructions and Ingredients

### Directions:

Contents arranged in sequence:

1. Unfold CSR wrap and use as a sterile field
2. Don gloves
3. Pull catheter out of tube to desired length. Lay tube on sterile field.
4. Prepare patient and proceed with catheterization. (**Note:** If urine does not flow freely into the specimen tube, loosen the cap slightly.)
5. After sample collection, pull catheter out of top; tighten cover and depress spout.
6. Send to lab in normal manner.

### Aplicare Three Povidone-Iodine Swabsticks

<b>Drug Facts</b>	
<b>Active ingredient</b>	<b>Purpose</b>
Povidone-iodine USP 10% .....	Antiseptic
<b>Use</b>	
Antiseptic skin preparation	
<b>Warnings</b>	
Do not use:	
<ul style="list-style-type: none"> <li>• If allergic to iodine</li> <li>• In the eyes</li> </ul>	
For external use only	
Ask a doctor before use if injuries are:	
<ul style="list-style-type: none"> <li>• Deep or puncture wounds</li> <li>• Serious burns</li> </ul>	
Stop use and ask a doctor if:	
<ul style="list-style-type: none"> <li>• Redness, irritation, swelling, or pain persists or increases</li> <li>• Infection occurs</li> </ul>	
Avoid pooling beneath patient	
<b>Keep out of reach of children.</b> In case of accidental ingestion, seek professional assistance or consult a poison control center immediately.	
<b>Directions</b>	
Apply locally as needed	
<b>Other information</b>	
<ul style="list-style-type: none"> <li>• 1% titratable iodine</li> <li>• Latex free</li> <li>• For hospital or professional use only</li> <li>• STERILE unless opened or damaged</li> </ul>	
<b>Inactive ingredients</b>	
Citric acid, disodium phosphate, nonoxynol-9, sodium hydroxide, water	

### Dover Lubricating Jelly 5g

- Water-soluble lubricant.
- For hospital and professional use.
- Will not harm surgical instruments, rubber, or plastic.
- Non-irritating to skin, tissue, and mucous membranes.
- Do not use if package is opened or damaged.