

Monitoring

⚠ WARNING Be sure to follow your facility's policies and guidelines for frequency of patient monitoring. Posey recommends that this product be removed at least every two (2) hours to check for skin integrity, proper circulation and range of motion.

Inspection

⚠ WARNING Inspect before each use: check for broken stitches or parts; or torn, cut or frayed material; DO NOT use soiled or damaged products.

Arm Application Instructions:

1. Beginning with upper arm opening, roll inside edge of SkinSleeve outwards; continue to roll until you reach the thumb opening.
2. Slip the arm opening of the SkinSleeve over the patient's hand, fitting the patient's thumb through the thumb hole.
3. Unroll the SkinSleeve back up over the arm. Adjust to desired placement.
4. If desired, the patient's thumb can be removed from the thumb opening and the SkinSleeve "rolled up" on the wrist to uncover the palm.




Leg Application Instructions:

1. Beginning with upper leg opening, roll inside edge of SkinSleeve outwards until you get within 3" of the toe opening.
2. Slip the upper leg opening of the SkinSleeve over the patient's foot, pulling it up until the heel "crook" of the SkinSleeve fits snugly over the patient's heel.
3. Unroll the SkinSleeve up the leg. Adjust to desired placement.

⚠ WARNING Inspect skin frequently for tears, lacerations, avulsions, abrasions, infection or other damage and determine appropriate treatment. Change product with new or laundered product in accordance with facility policy.

To Launder:

Machine wash and dry low temperature. For best results, hang dry.

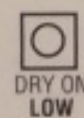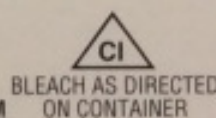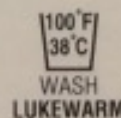

Disposal

⚠ WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.