

Fitting Guidelines

SkinSleeves are intended to fit as a comfort product. Actual fit will depend on individual anthropometric measurements and patient comfort level. Posey regular SkinSleeves will stretch up to 100% in size (combination of both width and length). Padded sizes have limited stretchability.

CAUTION The higher the degree of stretching, the tighter the fit and the higher amount of compression on the patient's skin and muscle.

SIZE/COLOR BAND	LENGTH	RECOMMENDED ARM CIRCUM. FOR A COMFORT FIT
Arm - Small/Red	15½" (39 ½ cm)	8½" (21½ cm)
Arm - Medium/Green	16½" (42 cm)	11" (28 cm)
Arm - Large/Yellow	18" (46 cm)	11" (28 cm)
Arm - X-Large/Blue	20" (51 cm)	13" (33 cm)
Leg - Medium/Green	13½" (34 cm)	11" (28 cm)

Monitoring

WARNING Be sure to follow your facility's policies and guidelines for frequency of patient monitoring. Posey recommends that this product be removed at least every two (2) hours to check for skin integrity, proper circulation and range of motion.

Inspection

WARNING Inspect before each use: check for broken stitches or parts; or torn, cut or frayed material; DO NOT use soiled or damaged products.

Arm Application Instructions:

1. Beginning with upper arm opening, roll inside edge of SkinSleeve outwards; continue to roll until you reach the thumb opening.
2. Slip the arm opening of the SkinSleeve over the patient's hand, fitting the patient's thumb through the thumb hole.
3. Unroll the SkinSleeve back up over the arm. Adjust to desired placement.
4. If desired, the patient's thumb can be removed from the thumb opening and the SkinSleeve "rolled up" on the wrist to uncover the palm.

Leg Application Instructions:

1. Beginning with upper leg opening, roll inside edge of SkinSleeve outwards until you get within 3" of the toe opening.
2. Slip the upper leg opening of the SkinSleeve over the patient's foot, pulling it up until the heel "crook" of the SkinSleeve fits snugly over the patient's heel.
3. Unroll the SkinSleeve up the leg. Adjust to desired placement.

WARNING Inspect skin frequently for tears, lacerations, avulsions, abrasions, infection or other damage and determine appropriate treatment. Change product with new or laundered product in accordance with facility policy.

To Launder:

Machine wash and dry low temperature. For best results, hang dry.


Disposal

WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

Storage And Handling

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

Posey Company • 5635 Peck Road • Arcadia, CA 91006-0020 USA • Tel: 1.800.447.6739
 or 1.626.443.3143 • Fax: 1.800.767.3933 or 1.626.443.5014 • www.posey.com

MADE IN TAIWAN