

[Print](#) [Close](#)

OPSITE[◇] FLEXIFIX[◇]

Transparent Film Roll

[Overview](#)[Instructions for use](#)

Step 1

Cut off the desired length of film from roll.

Step 2

Fold the strip along its length to crack the protective papers. Remove the backing paper marked No.1 "remove first".

Step 3

Apply the adhesive film to desired area and smooth down.

Step 4

Remove the green squared grid carrier, marked No 3.

to the skin and lift off.

To remove the film, gently stretch the film parallel

2010 SMITH & NEPHEW
